



Thin-layer Rapid Use Epicutaneous Patch Test

INFORMATION FOR PATIENTS

PATIENT NAME: _____

Patch testing will begin at: _____ (Time) on _____ (Day/Date)

Return for **reading #1** at: _____ (Time) on _____ (Day/Date)

Return for **reading #2** at: _____ (Time) on _____ (Day/Date)

Return for **reading #3** at: _____ (Time) on _____ (Day/Date)

Final results and counseling at: _____ (Time) on _____ (Day/Date)

Pre-test Instructions (if you have any questions, ask your doctor):

- Wear comfortable clothing
- No oral corticosteroids for 2 weeks prior to testing
- No skin lotions or medicines on test areas
- Shave off excessive back hair with an electric razor
- Avoid using alcohol or other irritating substances
- Other: _____

QUESTIONS AND ANSWERS ABOUT T.R.U.E. TEST[®] (ALLERGEN PATCH TEST)

What is allergic contact dermatitis? This is a skin reaction that occurs when you touch or contact substances that you are allergic to. Your skin can be itchy, cracked, red, sore, and even bleed. The substances that cause this reaction can be an ingredient in your makeup, aftershave, shampoo, jewelry, medication, and clothing. You may also find these substances at work in your cleaning supplies, paper and ink, medicines, disinfectants, construction materials and rubber products.



What is T.R.U.E. TEST? T.R.U.E. TEST is a reliable, easy to use patch test. It is designed to help your doctor find out whether you are allergic to the substances included on the test panels (shown in the table below). The test panels contain 35 different substances known to cause allergic contact dermatitis. T.R.U.E. TEST also includes a negative control that can help your doctor interpret your skin reactions.

Panel 1.2	Panel 2.2	Panel 3.2
1. Nickel sulphate	13. <i>p</i> -tert-Butylphenol formaldehyde resin	25. Diazolidinyl Urea
2. Wool alcohols	14. Epoxy resin	26. Quinoline Mix
3. Neomycin sulfate	15. Carba mix	27. Tixocortol-21-Pivalate
4. Potassium dichromate	16. Black rubber mix	28. Gold Sodium Thiosulfate
5. Caine mix	17. Cl+Me- Isothiazolinone	29. Imidazolidinyl Urea
6. Fragrance mix	18. Quaternium-16	30. Budesonide
7. Colophony	19. Methyl dibromo Glutaronitrile	31. Hydrocortisone-17-Butyrate
8. Paraben mix	20. <i>p</i> -Phenylenediamine	32. Mercaptobenzothiazole
9. Negative control	21. Formaldehyde	33. Bacitracin
10. Balsam of Peru	22. Mercapto mix	34. Parthenolide
11. Ethylenediamine dihydrochloride	23. Thimerosal	35. Disperse Blue 106
12. Cobalt dichloride	24. Thiuram mix	36. 2-Bromo-2-nitropropane-1,3-diol

How long will I have to wear T.R.U.E. TEST? You must wear the panels (on your back or an alternative site) for about 2 days. Do not remove the panels unless your doctor tells you to do so. You must return to have the panels removed. At this time, your doctor will take the first skin reading.

What activities should I avoid while wearing T.R.U.E. TEST? You must be careful NOT to get T.R.U.E. TEST panels or the surrounding skin wet. This can cause the panels to loosen, wash away the test substances or marking ink. To avoid this:

- **Do not wet T.R.U.E. TEST panels or the surrounding skin while bathing.** Take sponge baths for the first 48 hours and don't shower until the patches are removed.
- Avoid getting T.R.U.E. TEST panels or the surrounding skin wet while exercising or participating in activities that may cause you to sweat, such as vigorous exercise or sunbathing.



If an area of the test panel does become loose, use hypoallergenic adhesive tape to reattach them to your skin in exactly the same position as before. Apply tape only around the outside edge of the panels.

What should I do if my skin itches or burns while wearing T.R.U.E. TEST? Itching and burning sensations are common side effects. Try not to scratch the patch test area. Scratching can irritate your skin, and may make the itching worse. It also can affect your doctor's ability to interpret your skin's reaction. If the itching or burning becomes severe, you should contact your doctor immediately.

What reactions will alert my doctor? At each skin reading, your doctor will carefully examine the test area for signs of an allergic reaction. It may look like a small skin rash with swelling, redness and tiny blisters.

It is very important that you keep all appointments. Your doctor needs to see your skin and check for signs of an allergic reaction in the test area.



When will my T.R.U.E. TEST results be known? Some reactions to the substances in T.R.U.E. TEST appear within a few days, while others can take as long as 10 days to appear. Your doctor will determine how long it will take to complete your test based on your history and symptoms. Your doctor will discuss your results with you during your last appointment.

If I am allergic to a substance in T.R.U.E. TEST, what should I do? If you test positive, your doctor will explain which substance you are allergic to and talk to you about how to avoid contact. Your doctor will provide information about:

- 1) Where the substance can be found at work and at home;
- 2) What products are likely to contain this substance;
- 3) Steps you can take to avoid this substance; and
- 4) Alternative products you can use that don't contain this substance.

What does a negative test result mean? Negative results are very common. If you test negative, you will not have to avoid products that contain the 35 common allergens and allergen mixes included in T.R.U.E. TEST. However, you still can have other allergies. Although the 35 allergens and allergen mixes in T.R.U.E. TEST are the most common, there are many more substances that could be causing your symptoms. Some patients need additional tests to determine if they are allergic to less common allergens.

Your doctor may decide that your symptoms are due to an irritant reaction, and talk with you about how to better care for your skin and avoid irritating substances. If your doctor believes your skin's reaction is due to some other condition, you may be referred to a specialist for further testing and treatment.